
IDK[®] Zonulin ELISA

Zur in-vitro-Bestimmung von Zonulin in Serum
For the in vitro determination of zonulin in serum

Gültig ab / Valid from 2018-05-22

REF K 5601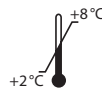

Immundiagnostik AG, Stubenwald-Allee 8a, 64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: + 49 6251 849430

e.mail: info@immundiagnostik.com

www.immundiagnostik.com

Inhalt

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15. LITERATUR

1. Fasano, A, T Not, W Wang, S Uzzau, I Berti, A Tommasini, and S E Goldblum. 2000. "Zonulin, a Newly Discovered Modulator of Intestinal Permeability, and Its Expression in Coeliac Disease." *Lancet* **355** (9214) (April 29): 1518–9. doi:10.1016/S0140-6736(00)02169-3.
2. Wang, W, S Uzzau, S E Goldblum, and A Fasano. 2000. "Human Zonulin, a Potential Modulator of Intestinal Tight Junctions." *Journal of Cell Science* **113** Pt 24 (December): 4435–40.
3. Fasano, A. 2001. "Intestinal Zonulin: Open Sesame!" *Gut* **49** (2) (August): 159–62.
4. Freemark, Michael, and Lynne L Levitsky. 2003. "Screening for Celiac Disease in Children with Type 1 Diabetes: Two Views of the Controversy." *Diabetes Care* **26** (6) (June): 1932–9.
5. Lazzarotto, Francesca, Daniela Basso, Mario Plebani, Alessandro Moscon, Renato Zanchetta, and Corrado Betterle. 2003. "Celiac Disease and Type 1 Diabetes." *Diabetes Care* **26** (1) (January): 248–9.
6. Watts, Tammara, Irene Berti, Anna Sapone, Tania Gerarduzzi, Tarcisio Not, Ronald Zielke, and Alessio Fasano. 2005. "Role of the Intestinal Tight Junction Modulator Zonulin in the Pathogenesis of Type I Diabetes in BB Diabetic-Prone Rats." *Proceedings of the National Academy of Sciences of the United States of America* **102** (8) (February 22): 2916–21. doi:10.1073/pnas.0500178102.
7. De Magistris, Maria Teresa. 2006. "Zonula Occludens Toxin as a New Promising Adjuvant for Mucosal Vaccines." *Vaccine* **24** Suppl 2 (April 12): S2–60–1.

8. Sapone, Anna, Laura de Magistris, Michelle Pietzak, Maria G Clemente, Amit Tripathi, Francesco Cucca, Rosanna Lampis, et al. 2006. "Zonulin Upregulation Is Associated with Increased Gut Permeability in Subjects with Type 1 Diabetes and Their Relatives." *Diabetes* **55** (5) (May 1): 1443–9. doi:55/5/1443 [pii].
9. Thomas, Karen E, Anna Sapone, Alessio Fasano, and Stefanie N Vogel. 2006. "Gliadin Stimulation of Murine Macrophage Inflammatory Gene Expression and Intestinal Permeability Are MyD88-Dependent: Role of the Innate Immune Response in Celiac Disease." *Journal of Immunology* (Baltimore, Md. : 1950) **176** (4) (February 15): 2512–21.

Verwendete Symbole:

Temperaturbegrenzung

Bestellnummer

In-Vitro-Diagnostikum

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Hersteller

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Achtung

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
IDK[®] Zonulin ELISA

For the in vitro determination of zonulin in serum

Valid from 2018-05-22

REF K 5601

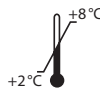IVD **CE**

Immundiagnostik AG, Stubenwald-Allee 8a, 64625 Bensheim, Germany

Tel.: +49 6251 70190-0

Fax: + 49 6251 849430

e.mail: info@immundiagnostik.com www.immundiagnostik.com

Table of Contents

1. INTENDED USE	17
2. INTRODUCTION	17
3. MATERIAL SUPPLIED	17
4. MATERIAL REQUIRED BUT NOT SUPPLIED	18
5. PREPARATION AND STORAGE OF REAGENTS	18
6. SPECIMEN COLLECTION AND PREPARATION	19
<i>Sample preparation</i>	19
<i>Preparation of standards and controls</i>	19
7. ASSAY PROCEDURE	20
<i>Principle of the test</i>	20
<i>Test procedure</i>	20
8. RESULTS	21
9. LIMITATIONS	22
10. QUALITY CONTROL	22
<i>Reference range</i>	23
11. PERFORMANCE CHARACTERISTICS	23
<i>Accuracy – Precision</i>	23
<i>Accuracy – Trueness</i>	23
<i>Linearity</i>	24
<i>Analytical sensitivity</i>	24
<i>Analytical specificity</i>	25
12. PRECAUTIONS	25
13. TECHNICAL HINTS	25
14. GENERAL NOTES ON THE TEST AND TEST PROCEDURE	26
15. REFERENCES	26

1. INTENDED USE

This ELISA is intended for the quantitative determination of zonulin in serum. For *in vitro* diagnostic use only.

2. INTRODUCTION

Zonulin is a novel human protein analogue to the zonula occludens toxin derived from *Vibrio cholerae* which participates in tight junctions between cells of the wall of the digestive tract. Zonulin binds to a specific receptor on the surface of intestinal epithelia and triggers a cascade of biochemical events which induces tight junction disassembly and a subsequent permeability increase of the intestinal epithelia, allowing some substances to pass through and activate immune reactions.

Dr. Fasano and his co-workers found out that the zonulin-zonulin-receptor-system is more activated in celiac disease and type 1 diabetes mellitus patients. Patients with active celiac disease showed higher levels of zonulin and anti-zonulin antibodies compared to non-celiac patients and patients in remission, who were on a gluten-free diet.

Concerning the autoimmune type 1 diabetes, in experiments with rats it could be demonstrated that elevated zonulin levels as well as increased intestinal permeability precede a type 1 diabetes disease. Conversely, type 1 diabetes could be prevented by inhibition of zonulin in animal experiments.

In addition, it was reported that many people who suffer from celiac disease also suffer from other autoimmune disorders. It is suggested that increased levels of zonulin are a contributing factor to the development of celiac disease and other autoimmune disorders such as insulin dependent diabetes, multiple sclerosis and rheumatoid arthritis.

3. MATERIAL SUPPLIED

Cat. No.	Label	Kit components	Quantity
K 5601	PLATE	Microtiter plate, pre-coated	12 x 8 wells
K 5601	WASHBUF	Wash buffer concentrate, 10x	2 x 100 ml
K 5601	DIL	Dilution buffer, ready-to-use	1 x 100 ml
K 5601	TRACER	Tracer concentrate, biotinylated zonulin	1 x 300 µl
K 5601	CONJ	Conjugate concentrate, peroxidase-labelled streptavidin	1 x 200 µl

Cat. No.	Label	Kit components	Quantity
K 5601	STD	Standards, lyophilised (see specification for concentrations)	4 x 5 vials
K 5601	CTRL1	Control, lyophilised (see specification for range)	4 x 1 vial
K 5601	CTRL2	Control, lyophilised (see specification for range)	4 x 1 vial
K 5601	SUB	Substrate (Tetramethylbenzidine), ready-to-use	1 x 15 ml
K 5601	STOP	Stop solution, ready-to-use	1 x 15 ml

For reorders of single components, use the catalogue number followed by the label as product number.

4. MATERIAL REQUIRED BUT NOT SUPPLIED

- Ultra pure water*
- Calibrated precision pipettors and 10–1000 µl tips
- Foil to cover the microtiter plate
- Horizontal microtiter plate shaker
- Multi-channel pipets or repeater pipets
- Vortex
- Standard laboratory glass or plastic vials, cups, etc. made of polypropylene
- Microtiter plate reader (required filters see chapter 7)

* Immundiagnostik AG recommends the use of Ultra Pure Water (Water Type 1; ISO 3696), which is free of undissolved and colloidal ions and organic molecules (free of particles > 0.2 µm) with an electrical conductivity of 0.055 µS/cm at 25 °C (≥ 18.2 MΩcm).

5. PREPARATION AND STORAGE OF REAGENTS

- To run the assay more than once, ensure that reagents are stored at the conditions stated on the label. **Prepare only the appropriate amount necessary for each run.** The kit can be used up to 4 times within the expiry date stated on the label.
- Reagents with a **volume less than 100 µl** should be centrifuged before use to avoid loss of volume.
- **Preparation of the wash buffer:** The **wash buffer concentrate (WASHBUF)** has to be diluted with ultra pure water **1:10** before use (100 ml WASHBUF + 900 ml ultra pure water), mix well. Crystals could occur due to high salt con-

centration in the concentrate. Before dilution, the crystals have to be redissolved at room temperature or in a water bath at 37°C. The **WASHBUF** is stable at **2–8°C** until the expiry date stated on the label. **Wash buffer** (1:10 diluted WASHBUF) can be stored in a closed flask at **2–8°C for 1 month**.

- The **lyophilised standards (STD)** and **controls (CTRL)** are stable at **2–8°C** until the expiry date stated on the label. **Reconstitution** details are given in the **specification data sheet**. **Standards and controls** (reconstituted STD and CTRL) **are not stable and cannot be stored**.
- **Preparation of the tracer:** The **tracer concentrate (TRACER)** has to be diluted **1:101 in dilution buffer** (e.g. 150 µl TRACER + 15 ml DIL) immediately before use. The TRACER is stable at **2–8°C** until expiry date given on the label. **Tracer** (1:101 diluted TRACER) **is not stable and cannot be stored**.
- **Preparation of the conjugate:** Immediately before use, the **conjugate concentrate (CONJ)** has to be diluted **1:101** in dilution buffer (e.g. 100 µl CONJ + 10 ml DIL). The CONJ is stable at **2–8°C** until expiry date stated on the label. **Conjugate** (1:101 diluted CONJ) **is not stable and cannot be stored**.
- All other test reagents are ready to use. Test reagents are stable until the expiry date (see label) when stored at **2–8°C**.

6. SPECIMEN COLLECTION AND PREPARATION

Zonulin is stable in undiluted serum for 12 months at -80°C as well as for 8 weeks at -20°C and for 1 day at 2–8°C. Zonulin is not stable at room temperature.

Sample preparation

1.	Pipet each 25 µl of serum samples in the respectively labelled reaction tubes.
2.	Add 475 µl of dilution buffer to each sample, vortex well. This results in a dilution factor of 20.
3.	Pipet 150 µl of diluted sample in labelled reaction tubes and add 150 µl of tracer to each sample, vortex well.

Preparation of standards and controls

Transfer **150 µl** of each **standard** or **control** in the correspondingly labelled reaction tubes, add **150 µl** of **tracer** and mix well.

7. ASSAY PROCEDURE

Principle of the test

This ELISA is designed for the quantitative determination of zonulin in serum samples.

This assay is based on the method of competitive ELISA. As a first preparation step, a biotinylated zonulin tracer is added to the samples, standards and controls. Afterwards, aliquots of the treated samples, standards and controls are transferred and incubated in microtiter plate wells coated with polyclonal anti-zonulin antibodies. During the incubation, the free target antigen in the samples competes with the biotinylated zonulin tracer for the binding of the polyclonal anti-zonulin antibodies immobilised on the microtiter plate wells. The unbound components are removed by a washing step. During a second incubation step, peroxidase-labelled streptavidin, which binds to the biotinylated zonulin tracer, is added into each microtiter well. After a washing step to remove the unbound components, the peroxidase substrate tetramethylbenzidine (TMB) is added. Finally, the enzymatic reaction is terminated by an acidic stop solution. The colour changes from blue to yellow and the absorbance is measured in the photometer at 450 nm. The intensity of the yellow colour is inverse proportional to the zonulin concentration in the sample; this means, high zonulin concentration in the sample reduces the concentration of the biotinylated zonulin tracer bound to the immobilised anti-zonulin antibodies and lowers the photometric signal. A dose response curve of absorbance unit (optical density, OD at 450 nm) vs. concentration is generated, using the values obtained from the standard. Zonulin, present in the patient samples, is determined directly from this curve.

Test procedure

Prior to use, allow all reagents and samples to come to room temperature (15–30°C) and mix well.

Mark the positions of standards/controls/samples on a protocol sheet.

Take as many microtiter strips (PLATE) as needed from kit. Store unused strips in the aluminium wrapper at 2–8°C. Strips are stable until the expiry date stated on the label.

For automated ELISA processors, the given protocol may need to be adjusted according to the specific features of the respective automated platform. For further details please contact your supplier or Immundiagnostik AG.

We recommend to carry out the tests in duplicate.

1.	Add each 100 µl of the prepared standards/controls/samples into the respective wells.
2.	Cover the strips and incubate for 1 hour shaking on a horizontal shaker at 550 rpm with an orbit of 2 mm at room temperature (15–30 °C).
3.	Discard the content of each well and wash 5 times with 250 µl wash buffer . After the final washing step, remove residual wash buffer by firmly tapping the plate on absorbent paper.
4.	Add 100 µl conjugate (diluted CONJ) into each well.
5.	Cover the strips and incubate for 1 hour shaking on a horizontal shaker at 550 rpm with an orbit of 2 mm at room temperature (15–30 °C).
6.	Discard the content of each well and wash 5 times with 250 µl wash buffer . After the final washing step, remove residual wash buffer by firmly tapping the plate on absorbent paper.
7.	Add 100 µl substrate (SUB) into each well.
8.	Incubate for 10–20 minutes at room temperature (15–30 °C)*.
9.	Add 100 µl stop solution (STOP) into each well and mix well.
10.	Determine absorption immediately with an ELISA reader at 450 nm against 620 nm (or 690 nm) as a reference. If no reference wavelength is available, read only at 450 nm. If the extinction of a sample or a standard exceeds the range of the photometer, absorption must be measured immediately at 405 nm against 620 nm as a reference.

* The intensity of the colour change is temperature sensitive. We recommend observing the colour change and stopping the reaction upon good differentiation.

8. RESULTS

The following algorithms can be used alternatively to calculate the results. We recommend using the “4 parameter algorithm”.

1. 4 parameter algorithm

It is recommended to use a linear ordinate for the optical density and a logarithmic abscissa for the concentration. When using a logarithmic abscissa, the zero standard must be specified with a value less than 1 (e.g. 0.001).

2. Point-to-point calculation

We recommend a linear ordinate for the optical density and a linear abscissa for the concentration.

3. Spline algorithm

We recommend a linear ordinate for the optical density and a linear abscissa for the concentration.

The plausibility of the duplicate values should be examined before the automatic evaluation of the results. If this option is not available with the programme used, the duplicate values should be evaluated manually.

Serum samples

The obtained results have to be multiplied with the **dilution factor of 20** to get the actual concentrations.

In case **another dilution factor** has been used, multiply the obtained result with the dilution factor used.

9. LIMITATIONS

Samples with concentrations above the measurement range (see definition below) must be further diluted and re-assayed. Please consider this greater dilution when calculating the results.

Samples with concentrations lower than the measurement (see definition below) range cannot be clearly quantified.

The upper limit of the measurement range can be calculated as:

highest concentration of the standard curve × sample dilution factor to be used

The lower limit of the measurement range can be calculated as:

LoB × sample dilution factor to be used

10. QUALITY CONTROL

Immundiagnostik recommends the use of external controls for internal quality control, if possible.

Control samples should be analysed with each run. Results, generated from the analysis of control samples, should be evaluated for acceptability using appropriate statistical methods. The results for the patient samples may not be valid if within the same assay one or more values of the quality control sample are outside the acceptable limits.

Reference range

Based on Immundiagnostik studies of serum samples of apparently healthy persons (n = 40), a median value of 34 ng/ml (\pm 14 ng/ml) was estimated.

We recommend each laboratory to establish its own reference range.

Establish own reference ranges for plasma samples.

11. PERFORMANCE CHARACTERISTICS

Accuracy – Precision

Repeatability (Intra-Assay); n=40

The repeatability was assessed with 2 serum samples under constant parameters (same operator, measurement system, day and kit lot).

Sample	Mean value [ng/ml]	CV [%]
1	43.90	3.5
2	38.38	6.0

Reproducibility (Inter-Assay); n=25

The reproducibility was assessed with 2 serum samples under varying parameters (different operators, measurement systems, days and kit lots).

Sample	Mean value [ng/ml]	CV [%]
1	41.13	7.7
2	46.15	8.3

Accuracy – Trueness

The trueness states the closeness of the agreement between the result of a measurement and the true value of the measurand. Therefore, zonulin spikes with known concentrations were added to 3 different serum samples.

Sample [ng/ml]	Spike [ng/ml]	Expected [ng/ml]	Obtained [ng/ml]	Recovery [%]
10.788	51.58	62.469	66.331	106.2
	37.71	48.494	50.259	103.6
	26.64	37.430	34.469	92.1

Sample [ng/ml]	Spike [ng/ml]	Expected [ng/ml]	Obtained [ng/ml]	Recovery [%]
12.428	51.58	64.109	70.666	110.2
	37.71	50.134	50.883	101.5
	26.64	39.070	33.032	84.5
13.372	51.58	65.053	70.547	108.4
	37.71	51.078	53.548	104.8
	26.64	40.014	34.266	85.6

Linearity

The linearity states the ability of a method to provide results proportional to the concentration of analyte in the test sample within a given range. This was assessed according to CLSI guideline EP6-A with a serial dilution of 2 different serum samples.

For zonulin in serum, the method has been demonstrated to be linear from 3.03–40.25 ng/ml, showing a non-linear behaviour of less than $\pm 20\%$ in this interval for concentrations greater than the Limit of Quantitation.

Sample	Dilution	Expected [ng/ml]	Obtained [ng/ml]	Recovery [%]
A	1:20	36.69	36.69	100.00
	1:40	18.34	19.02	103.70
	1:80	9.17	9.92	108.17
	1:160	4.59	5.91	128.79
B	1:20	40.25	40.25	100.00
	1:40	20.13	20.41	101.41
	1:80	10.06	10.90	108.29
	1:160	5.03	5.64	112.09
	1:320	2.52	3.03	120.52

Analytical sensitivity

The following values have been estimated based on the concentrations of the standard without considering possibly used sample dilution factors

Limit of blank, LoB	0.140 ng/ml
Limit of detection, LoD	0.183 ng/ml

Limit of quantitation, LoQ

0.183 ng/ml

The evaluation was performed according to the CLSI guideline EP17-A2. The specified accuracy goal for the LoQ was 20% CV.

Analytical specificity

The specificity of the antibody was tested by measuring the cross-reactivity against human haptoglobin. There was no cross-reactivity observed.

Substance tested	Concentration added	Concentration obtained	Conclusion
Human haptoglobin	2.9 mg/ml	< 0.02 ng/ml	< LoB

12. PRECAUTIONS

- All reagents in the kit package are for *in vitro* diagnostic use only.
- Human materials used in kit components were tested and found to be negative for HIV, Hepatitis B and Hepatitis C. However, for safety reasons, all kit components should be treated as potentially infectious.
- Kit reagents contain sodium azide or Proclin as bactericides. Sodium azide and Proclin are toxic. Substrates for the enzymatic colour reactions are toxic and carcinogenic. Avoid contact with skin or mucous membranes.
- The stop solution consists of diluted sulphuric acid, a strong acid. Although diluted, it still must be handled with care. It can cause burns and should be handled with gloves, eye protection, and appropriate protective clothing. Any spill should be wiped up immediately with copious quantities of water. Do not breath vapour and avoid inhalation.

13. TECHNICAL HINTS

- Do not interchange different lot numbers of any kit component within the same assay. Furthermore we recommend not assembling wells of different microtiter plates for analysis, even if they are of the same batch.
- Control samples should be analysed with each run.
- Reagents should not be used beyond the expiration date stated on kit label.
- Substrate solution should remain colourless until use.

- To ensure accurate results, proper adhesion of plate sealers during incubation steps is necessary.
- Avoid foaming when mixing reagents.
- Do not mix plugs and caps from different reagents.
- The assay should always be performed according to the enclosed manual.

14. GENERAL NOTES ON THE TEST AND TEST PROCEDURE


- This assay was produced and distributed according to the IVD guidelines of 98/79/EC.
- The guidelines for medical laboratories should be followed.
- IDK® is a trademark of Immundiagnostik AG.
- Incubation time, incubation temperature and pipetting volumes of the components are defined by the producer. Any variation of the test procedure, which is not coordinated with the producer, may influence the results of the test. Immundiagnostik AG can therefore not be held responsible for any damage resulting from incorrect use.
- Warranty claims and complaints regarding deficiencies must be logged within 14 days after receipt of the product. The product should be sent to Immundiagnostik AG along with a written complaint.

15. REFERENCES

1. Fasano, A, T Not, W Wang, S Uzzau, I Berti, A Tommasini, and S E Goldblum. 2000. "Zonulin, a Newly Discovered Modulator of Intestinal Permeability, and Its Expression in Coeliac Disease." *Lancet* **355** (9214) (April 29): 1518–9. doi:10.1016/S0140-6736(00)02169-3.
2. Wang, W, S Uzzau, S E Goldblum, and A Fasano. 2000. "Human Zonulin, a Potential Modulator of Intestinal Tight Junctions." *Journal of Cell Science* **113** Pt 24 (December): 4435–40.
3. Fasano, A. 2001. "Intestinal Zonulin: Open Sesame!" *Gut* **49** (2) (August): 159–62.
4. Freemark, Michael, and Lynne L Levitsky. 2003. "Screening for Celiac Disease in Children with Type 1 Diabetes: Two Views of the Controversy." *Diabetes Care* **26** (6) (June): 1932–9.

5. Lazzarotto, Francesca, Daniela Basso, Mario Plebani, Alessandro Moscon, Renato Zanchetta, and Corrado Betterle. 2003. "Celiac Disease and Type 1 Diabetes." *Diabetes Care* **26** (1) (January): 248–9.
6. Watts, Tammarra, Irene Berti, Anna Sapone, Tania Gerarduzzi, Tarcisio Not, Ronald Zielke, and Alessio Fasano. 2005. "Role of the Intestinal Tight Junction Modulator Zonulin in the Pathogenesis of Type I Diabetes in BB Diabetic-Prone Rats." *Proceedings of the National Academy of Sciences of the United States of America* **102** (8) (February 22): 2916–21. doi:10.1073/pnas.0500178102.
7. De Magistris, Maria Teresa. 2006. "Zonula Occludens Toxin as a New Promising Adjuvant for Mucosal Vaccines." *Vaccine* **24** Suppl 2 (April 12): S2–60–1.
8. Sapone, Anna, Laura de Magistris, Michelle Pietzak, Maria G Clemente, Amit Tripathi, Francesco Cucca, Rosanna Lampis, et al. 2006. "Zonulin Upregulation Is Associated with Increased Gut Permeability in Subjects with Type 1 Diabetes and Their Relatives." *Diabetes* **55** (5) (May 1): 1443–9. doi:55/5/1443 [pii].
9. Thomas, Karen E, Anna Sapone, Alessio Fasano, and Stefanie N Vogel. 2006. "Gliadin Stimulation of Murine Macrophage Inflammatory Gene Expression and Intestinal Permeability Are MyD88-Dependent: Role of the Innate Immune Response in Celiac Disease." *Journal of Immunology* (Baltimore, Md. : 1950) **176** (4) (February 15): 2512–21.

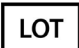Used symbols:

	Temperature limitation		Catalogue Number
	In Vitro Diagnostic Medical Device		To be used with
	Manufacturer		Contains sufficient for <n> tests
	Lot number		Use by
	Attention		Consult instructions for use